


Quick Reference Guide

Pro Focus 2202 Black & White



we specialize because you do

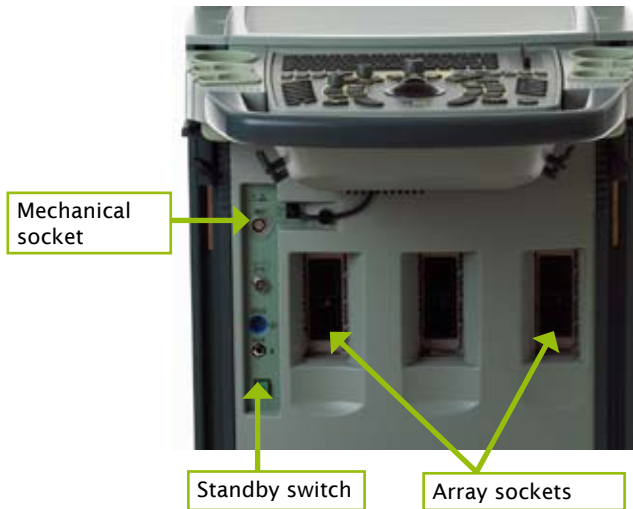

Turn Scanner On and Off

Use green standby switch.

To turn scanner on:

- Press ONCE. Wait until startup screen disappears. Scanner is ready to use.

To turn scanner off:

- Make sure scanner is completely up and running. Press standby switch ONCE.

Caution: *Never unplug scanner from wall while it is running! Turn off scanner and wait for light on switch to go out before unplugging.*

Connect Transducers

Array transducers

To connect:

1. Insert transducer plug into socket.
2. Turn locking lever on transducer plug clockwise.

To disconnect:

1. Turn locking lever on plug counterclockwise.
2. Remove plug from socket.

Mechanical transducers

To connect:

1. Align red mark on plug with red dot on transducer socket.
2. Insert plug in socket.

To disconnect:

1. Pull back on plug locking mechanism (collar).
2. Remove plug from socket.

Enter Patient Information

1. Press [F10 ID]. Patient window opens.

Worklist Update Today only

Date	Time	Name	Birth Date	ID	Sex	Description	Mo
------	------	------	------------	----	-----	-------------	----

Last First Mid

Patient ID

Keep image settings

Edit OK Cancel

2. Enter patient data. Use tab key to move from field to field.
3. Click **OK**.

Select Pro Package and Transducer Preset (Diagnostic Setup)

1. Press [F10].

Full Table Pro Package Only

Surgery	2050 H	10.0-16.0 MHz
---------	--------	---------------

Anal-1


Anal-2

Rectal-1

Rectal-2

Close

2. Select Pro Package and Diagnostic Setup.
3. Click **Close**.

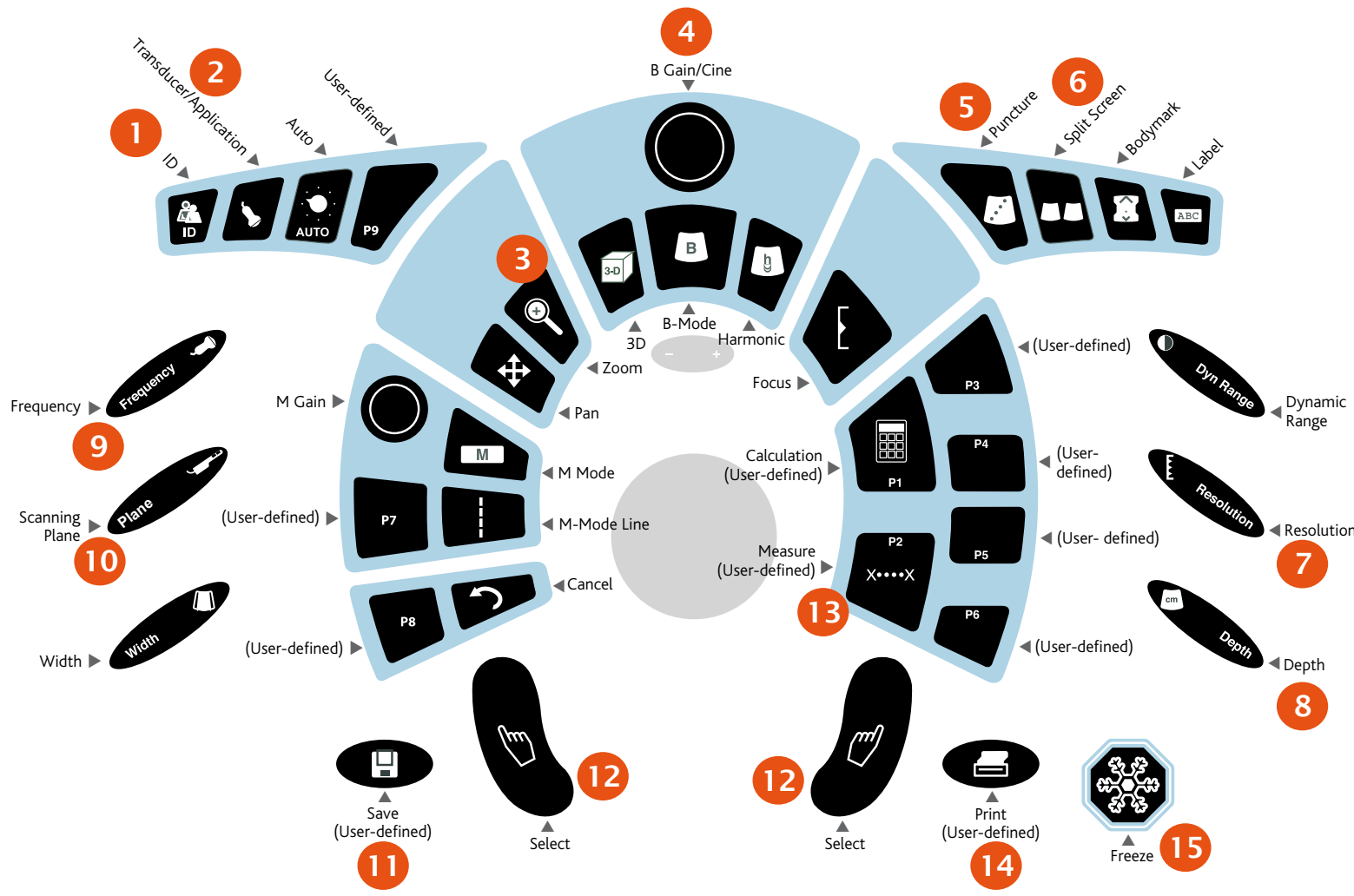

Frequently Used Keys

1. **ID:** Opens Patient window to start examination.
2. **Transducer/Application:** Press to select Pro Package or Diagnostic Setup.
3. **Zoom:** Turns on the zoom box. Press again to zoom in. A third press resets the zoom area. A long press turns zoom off if it is on.
Works when image is frozen or not frozen.
4. **Gain:** Brighter/Darker.
5. **Puncture:** Activates puncture lines/brachy matrix.
6. **Simultaneous/Split Screen:**
Activates simultaneous imaging.
Displays 2 screens.
7. **Resolution:** (Rocker key). Increases resolution (decreases scanning frame rate) or vice versa in B-mode.
8. **Depth:** (Rocker key). Changes maximum depth (distance from transducer). Press + to make objects appear larger, - to make them appear smaller.
9. **Frequency:** (Rocker key). Changes MHz.


10. **Plane:** (Rocker key). Changes Transverse/Sagittal.
11. **Save:** Stores image on hard drive.
12. **Select/Click:** Press to select option.
13. **Basic Measure:** Activates cursors for basic measurement.
14. **Print:** Sends image to printer.
15. **Freeze/Unfreeze**

View Exam

Use document browser at bottom of screen.



1. Click **View Exam**. Thumbnails of documents (including images) in exam appear in image bar.
2. Point at document you want to view.

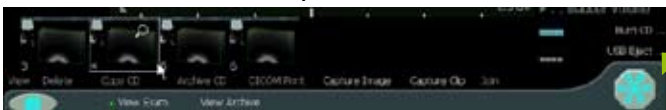

A magnifying glass icon appears in upper right corner of thumbnail.



3. Click magnifying glass. Little frame around it turns blue. Document is displayed in the scan area.

Copy to Flash

Images are copied to flash devices immediately.

1. Insert flash device into USB port.
2. Click each image to be copied. Frame around image turns blue.
3. Click **Copy Flash** or **Copy -ID Flash**.

(If copy option you want is not shown, point at the copy option shown, and click triangle for choices.)



4. Click **USB Eject** to remove flash.

Copy to CD

When you click **Copy CD**, the selected images are **NOT** copied immediately to a CD. They are copied to a staging area.


To copy:

1. Click each image to be copied. Frame around image turns blue.
2. Click **Copy CD** or **Copy -ID CD**.


(If copy option you want is not shown, point at copy option shown, and click triangle for choices.



3. Insert a CD into CD drive.

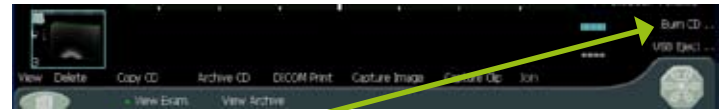

4. Click **Burn CD**.

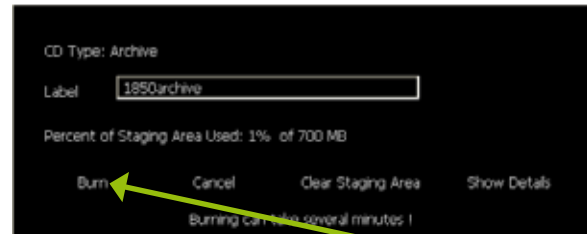

5. Name (label) CD and click **Burn**.

P Shortcut Keys

P1: 

P2: X••••X

P3:

P4:

P5:

P6:

P7:

P8:

P9:



F Shortcut Keys

F1:

F2:

F3:

F4:

F5:

F6:

F7:

F8: Report

F9:

F10: Add pointer arrow to image

Your Notes

The *Quick Reference Guide* is not a complete user guide. Before you use the scanner, you must read the *Pro Focus User Guide*. It contains information that is important for safety and for avoiding damage to the equipment.



BK Medical

World Headquarters
Mileparken 34
DK-2730 Herlev
Denmark

Tel: +45 4452 8100 · Fax: +45 4452 8199 · www.bkmed.com

With more than 30 years of commitment to ultrasound innovation, B-K Medical specializes in the development, manufacture and distribution of dedicated ultrasound solutions. B-K Medical headquarters are located near Copenhagen, Denmark, and we have offices and distributors throughout the world.

A wholly owned subsidiary of Analogic Corporation

Your local representative

Represented in

Argentina · Australia · Austria · Bangladesh · Belgium · Bolivia · Brazil · Bulgaria · Canada · Chile · P. R. of China · Colombia · Costa Rica · Croatia · Czech Republic · Denmark · Ecuador · Egypt · Estonia · Finland · France · Germany · Ghana · Greece · Guatemala · Hong Kong · Hungary · Iceland · India · Indonesia · Iran · Iraq · Ireland · Israel · Italy · Japan · Jordan · Kazakhstan · Korea · Kuwait · Latvia · Lebanon · Lithuania · Luxembourg · Malaysia · Malta · The Netherlands · New Zealand · Norway · Pakistan · Peru · Philippines · Poland · Portugal · Romania · Russia · Saudi Arabia · Serbia & Montenegro · Singapore · Slovak Republic · Slovenia · South Africa · Spain · Sweden · Switzerland · Syria · Taiwan · Thailand · Tunisia · Turkey · Ukraine · UAE · UK · Uruguay · USA · Venezuela · Vietnam

we specialize because you do