




New Histological Biopsy Needle Kit UA0039 for 8666/8566 and 8666-RF/8566-RF Transducers

Customer Information Note

Histological Biopsy Needle Kit UA0039 replaces UA0034 described in the user guide for the 8666 and 8666-RF transducers. (The new kit is also suitable for use with the 8566 and 8566-RF transducers.)

UA0039 comes with two flexible plastic tubes (needle guides) instead of the one black Pebax needle guide described in the user guide:

- a black needle guide to work with 8666 (and 8566)
- a blue needle guide to work with 8666-RF (and 8566-RF) – *but only with an updated version of biopsy attachment UA1249. The updated version is marked with an engraved oval next to the UA number (see picture below).*

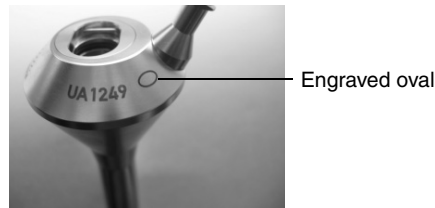

UA number and oval on updated version of biopsy attachment UA1249.

To use the new kit, follow the directions in the user guide in the section “Intraoperative Biopsy with UA0034”, but make sure that you use the correct needle guide (black or blue) for your transducer type.

WARNING Make sure that you do *NOT* use the black needle guide with the 8566-RF or 8666-RF transducer. The guide will fit in the hole, but the needle will not be fixated securely.

Caution: The blue needle guide can *ONLY* be used with the *updated* version of UA1249.

